

### Why and Who Should Test For HIV?

HIV is a virus that causes AIDS (Acquired Immune Deficiency Syndrome). Anyone can be at risk. Most infected people do not know their HIV status as they look and feel well. Most common routes of infection are:

- Unprotected sexual contacts with infected person(s)
- Repeated use of needles in connection with drugs
- Use of unsafe blood products or transfusion with infection blood
- Transfer of infection from a positive mother to her child
- Intravenous drug users
- Health care workers who take blood samples

### What is iCARE One Step HIV 1&2 Rapid Test Kit Used For?

iCARE One Step HIV 1&2 Rapid Test Kit will give the user a private, sensible, quick and accurate rapid test results in 5 minutes to ascertain if the user is infected with HIV virus. It is an aid only to identify infected individuals.

\*Please note that the Test Kit is an aid only to identify individual at this point of time. Doctors advise that the typical period of time between first contact and a positive test indication is about 90 days.

### When Should I Test?

If you have reason to suspect a possible infection of yourself or your sexual partner(s), it is wise to take a test. We advise you to wait 3 months before you do a test because although most people will manifest antibody levels within 30 days, a significant percentage do not show these anti-bodies until about 90 days. The National CDC has said that in some rare cases, it may even take up to six months for one to be tested positive. We recommend you do 2 tests 30 days' apart

### What If I Am Tested Positive?

If you are tested positive, please consult your doctor immediately.

### Each Pack of Test Kit Contains:

- Cassette
- Micro Pipette
- Alcohol Prep
- Buffer Bottle (Chemical)
- Lancet (Needle)
- Desiccant (Drying Agent – for keeping contents Dry)



### Disclaimer

Each test is for one time usage only. When performing the HIV test, the steps involved must be followed closely. If there has been a possible exposure to infected blood, and the person tests negative for HIV, the test be repeated in 90 days. It is imperative that a positive HIV test result be followed by a Western Blot or PCR test performed by a doctor or clinic to confirm if you are indeed HIV positive. Since NO test, nor test kit is infallible, it is best to confirm test results by using a licensed medical testing facility. For one time usage only.


## iCARE One Step HIV1&2 Rapid Test Kit Step by Step Instructions




1 Tear open the Alcohol Prep Pad




2 Clean your finger

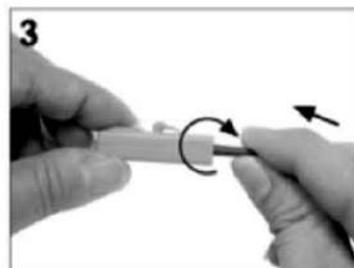


3 Push in and twist the knob of the Lancet

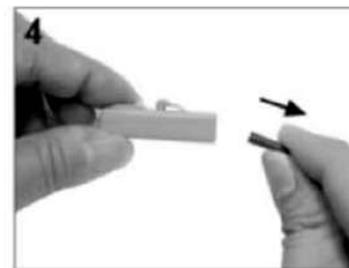

4 Pull out the knob

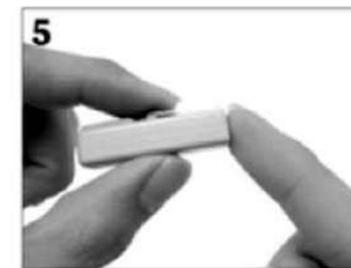

5 Place the Lancet at the finger and press the trigger to prick the finger

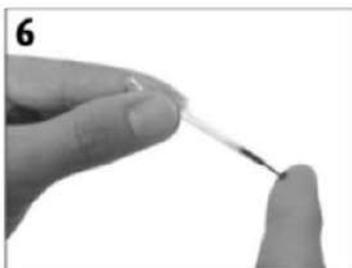

6 Collect 2 drops of blood samples using the micro pipette




7 Place 2 drops of blood sample into the Sample Well using the Micro Pipette



8 Open the cap of the Diluent Bottle



9 Place 2 drops of Diluent Liquid into the Sample Well slowly (Make sure the Liquid touches the Control Line)

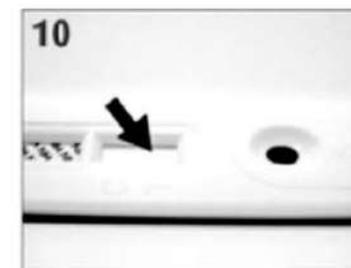


10 Read results within 15 minutes

**NOTE:**

When performing the HIV Test, the steps must be followed closely  
Use within 10 minutes after opening  
For Single Test Only  
Do not share the Lancet  
Dispose properly after use

### Interpretation of Result

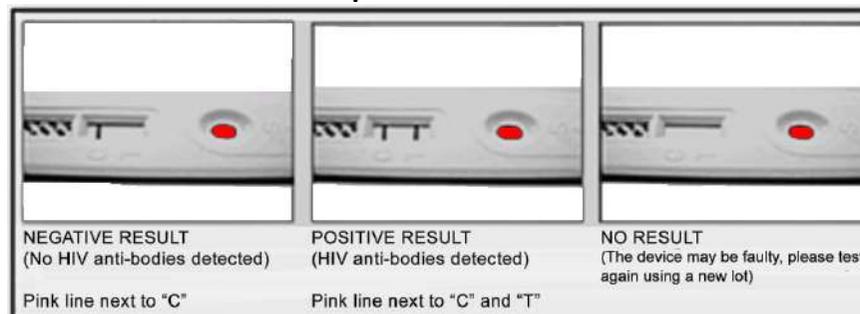

Product of JAL Innovation (Singapore) Pte Ltd  
190 Middle Road  
#19-05 Fortune Centre  
Email : [sales@jalinnovation.com](mailto:sales@jalinnovation.com)  
Website : [www.jalinnovation.com](http://www.jalinnovation.com)